

How to set up your new COR One™

The COR One™ contains a camera inside. It takes images of your blood sample periodically for 30 minutes after you insert your sample.

The COR One™ must transmit those images to our servers to calculate your inflammation. It must use your WiFi network for image upload.

Before using the COR One™ for the first time, you must configure it with your WiFi network name and password. You will need a computer or mobile phone to do so.

YOU MUST KNOW YOUR WIFI NETWORK NAME AND PASSWORD

To associate and register your COR One, you will configure your email address in the remaining steps below. The email address entered will be used to notify you when your inflammation results are ready. You will then use the same email address to log in and view your inflammation results in your dashboard and set up any additional family member dashboards. When registration is successful you will get an email with a link to the web app.

Note: please use a standard email and not a private server, so your results emails do not get blocked. gmail.com, aol.com, me.com, hotmail.com etc. work great.

Instructions:

1 **Power on** your COR One™ by attaching the supplied charger's USB-C cable to the USB-C socket on the rear of your COR One™. Then, connect the mains plug of the charger into your electrical outlet.

2 The COR One™ should now be emitting a **yellow light**. This signals that the unit is broadcasting a WiFi network.

3 On your device (phone, tablet, or computer), go to **WiFi settings** and connect to "CorOne_<your unique device serial number>". You may need to refresh your available WiFi list to see the network.

4 The COR One™ **WiFiManager screen** should open up automatically. If it does not, type the following address into your web browser <http://192.168.4.1/>

5 Select "**Configure WiFi**".

6 Select **your WiFi** network from the list.

7 Enter your **WiFi password** carefully into the field "Password".

Note: it's a good idea to power cycle the COR One before use. The device remembers your wifi. Device is ready when the light turns from initial yellow to blue.

Instructions:

8

Enter the **email address** carefully into the field "Email". This should be the email address that you want to be associated with device ownership. You will use this email later to log into COR One™ Dashboard to retrieve your inflammation results.

9

Select "**Save**".

10

The COR One™ should now be emitting a **blue light**. This signals that the unit has successfully connected to your WiFi network. Your phone, tablet, or computer will automatically switch back to your WiFi network.

Congratulations on successfully setting up your COR One™. If you have more than one network, maybe one at home and one at the office, you can repeat steps 3-7 to configure it with the second WiFi. The unit will remember multiple networks and connect automatically.

11

You will receive **an email** titled "COR One™ Dashboard Account Creation". We will send the email to the address you entered in Step 8. Please check Spam if you are still waiting to receive it after five minutes. Please follow the instructions in the email.

If you have any difficulties in setting up your COR One™, please:

If you still have any issues or questions, please feel free to contact us at support@corhealth.com

Note: you can also find these instructions (and instructional videos) on-line by going to:
corhealth.helpscoutdocs.com